

Downloading the Dana CamAPS FX App

When will Amazon Appstore stop working?

- ✓ After the 20th August 2025, you will no longer be able to download or update the CamAPS FX app from the Amazon AppStore.
- ✓ Your current CamAPS FX app on your Android phone will continue to work.

Where will I be able to download the app from?

- ✓ Dana CamAPS FX users, living in the UK, will be able to download the app from Google Play.
- ✓ Before you download the new app, please:
 1. Generate a PDF report (App's main menu > Statistics > Generate PDF report) so you have a copy of your App settings and Total Daily Dose (TDD).
 2. Check your body weight (App's main menu > Settings > Change Weight).
 3. Make a note of your Dexcom transmitter serial number (App's main menu).
 4. Delete the current app from your phone before attempting to download a new one.

How do I download the App from Google Play?

- ✓ Scan the QR code or
- ✓ Open your Google Play App on your compatible Android phone and search for the CamAPS FX app.



Will I lose my learning or settings?


- ✓ Your learning and app settings will be retained, if you log in with your same email.
- ✓ There is no need to create a new app account.
- ✓ Please check that your Alert, Share and Personal Glucose Target settings are correct.

Do Companions also have to update their apps?

- ✓ Yes, both the user and companion apps need to be downloaded from the same app store to allow data to flow between them.

Once the app has downloaded:

1. Open the app
2. Select your country of residence
3. Select Dana pump
4. Enter your email and password
5. Log into FX
6. Enter your weight and TDD
7. Pair your Dana pump and G6 transmitter
8. Start Auto mode

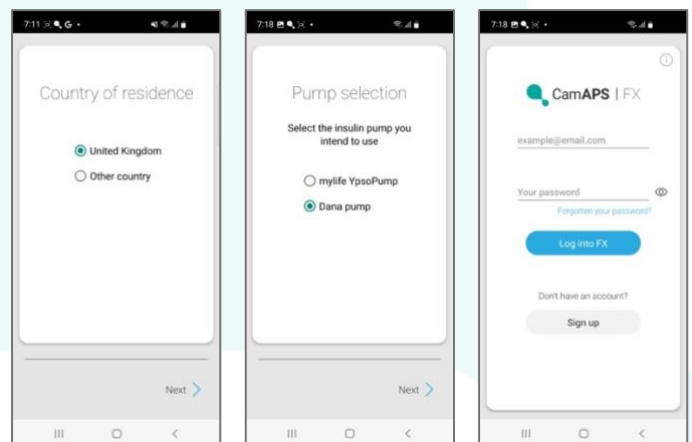

Please plan your changeover to Google Play carefully.

Choose a time in the week, when you can get support from your Diabetes team and Advanced Therapeutics (ATUK - email: info@atuk.ltd or call: [01926 833 273](tel:01926833273)), if necessary.