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IMPORTANT

Keep the instructions for future reference. Read these instructions before assembling and using this product. Adult assembly is required.

Thank you for choosing the TENSIMAX Upper Arm Blood Pressure Monitor! In this manual you will find all the information you need to assemble, use and maintain this product correctly and safely.

This blood pressure monitor has among its main functions, automatic measurement, an easy-to-read digital display, arrhythmia detection and 90 memories for storing various measurements.

Before you start, we recommend that you read this manual carefully to ensure optimal and safe use of the product.

As a guarantee of the quality of our products, the manufacturer has a management system certified under ISO 13485. And this product complies with the requirements of Regulation (EU) 2017/745 on medical devices.

Notice to the user and/or patient: any serious incident that has occurred in relation to the device must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is established.

1. WARNINGS AND PRECAUTIONS

- 1. WARNING!** The AC adapter cord and rubber air tube can cause accidental strangulation in infants.
- 2. WARNING!** Do not swallow small parts that may cause choking hazard.
- 3. WARNING!** Do not use the cuff on an injured arm or while on an intravenous drip.
- 4. WARNING!** Do not use the device if the arm where intravascular access or therapy, or an arteriovenous (A-V) shunt is present due to temporary interference with blood flow as it may cause injury to the patient.
- 5. WARNING!** Do not use the cuff on the arm of a mastectomy or lymph node cleansing.
- 6. WARNING!** Note that you will lose the function of other monitoring devices simultaneously on the same limb while the cuff is being inflated.
- 7. WARNING!** Do not allow frequent measurements. The resulting restriction of blood flow may cause injury.
- 8. WARNING!** Do not entangle the connecting tubing, as continuous pressure from the cuff may affect blood flow interference, causing harmful injury to the patient.

2. PURPOSES OF THE PRODUCT

- **Measuring blood pressure:** Check your blood pressure at any time.
- **Arrhythmia detection:** This model measures the pulse, thus detecting irregularities in the heart rhythm.
- **Portable and easy to use:** Compact, battery-powered and easy to operate without technical knowledge.
- **Save results:** Stores previous measurements for follow-up.


3. PRODUCT FEATURES

PRODUCT NAME	Electronic upper arm blood pressure monitor
MODEL	TENSIMAX
MODEL REFERENCE	E-00072/45
PRODUCT WEIGHT	<ul style="list-style-type: none"> • Sphygmomanometer dimensions: 5,11" * 3,77" * 2,95" • Cuff: 8,66"-17,71" (adults over 12 years)
MATERIALS	<ul style="list-style-type: none"> • ABS rear cover • PC top cover • Bracelet: Nylon and polyester • Air plug connector: ABS • Air tube: PVC
OTHER CHARACTERISTICS	<ul style="list-style-type: none"> • Capacity to store up to 90 measurements • Display of the average value of three measurements • Large LCD display • Irregular heartbeat indication • Sleeve wear detection • Motion error indication • Average of three measurements • Battery: 4 AA batteries. The blood pressure monitor can also be used without batteries, plugged in via USB


TECHNICAL PARAMETERS:

- Display: LCD digital display.
- Operating principle: Oscillometric method.
- Measurement range:
 - Diastolic: 20 mmHg - 210 mmHg.
 - Systolic: 40 mmHg - 260 mmHg.
 - Cuff pressure: 0 mmHg ~ 300 mmHg.
 - Pulse rate: 40 beats/min ~ 200 beats/min.
- Accuracy:
 - Pressure: within ± 3 mmHg (± 0.4 kPa.)
 - Pulse rate: within 5% of reading.
 - Working system: continuous operation.
- IP rating: IP20:
 - "0" No protection.
 - "2" against ingress of solid foreign bodies: $\geq 0,49$ " in diameter.
- Electrical classification: Class II and internal power supply, type BF applied part (sleeve is applied part).
- Food:
 - Battery: 4 AA batteries (not included).
 - USB for using the blood pressure monitor plugged in without batteries.
 - Battery life: the battery can be used up to 300 times.


4. PARTS AND COMPONENTS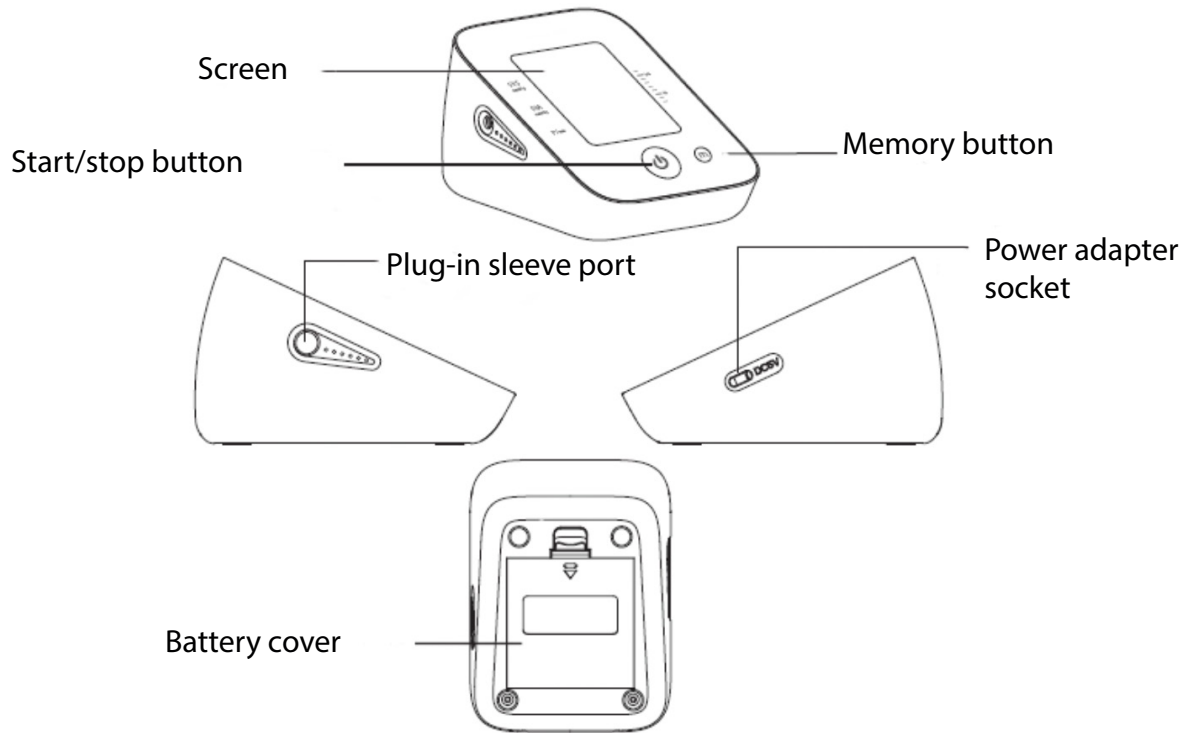


HANDLE

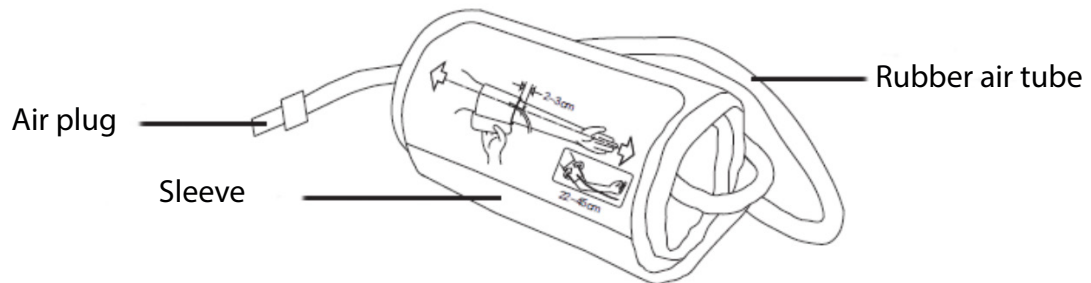USB CABLE

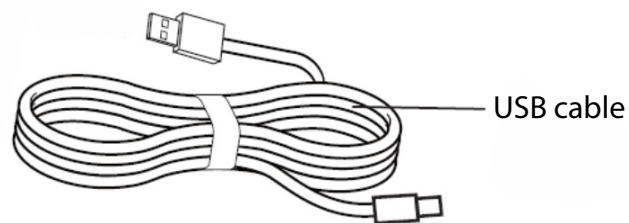DISPLAY

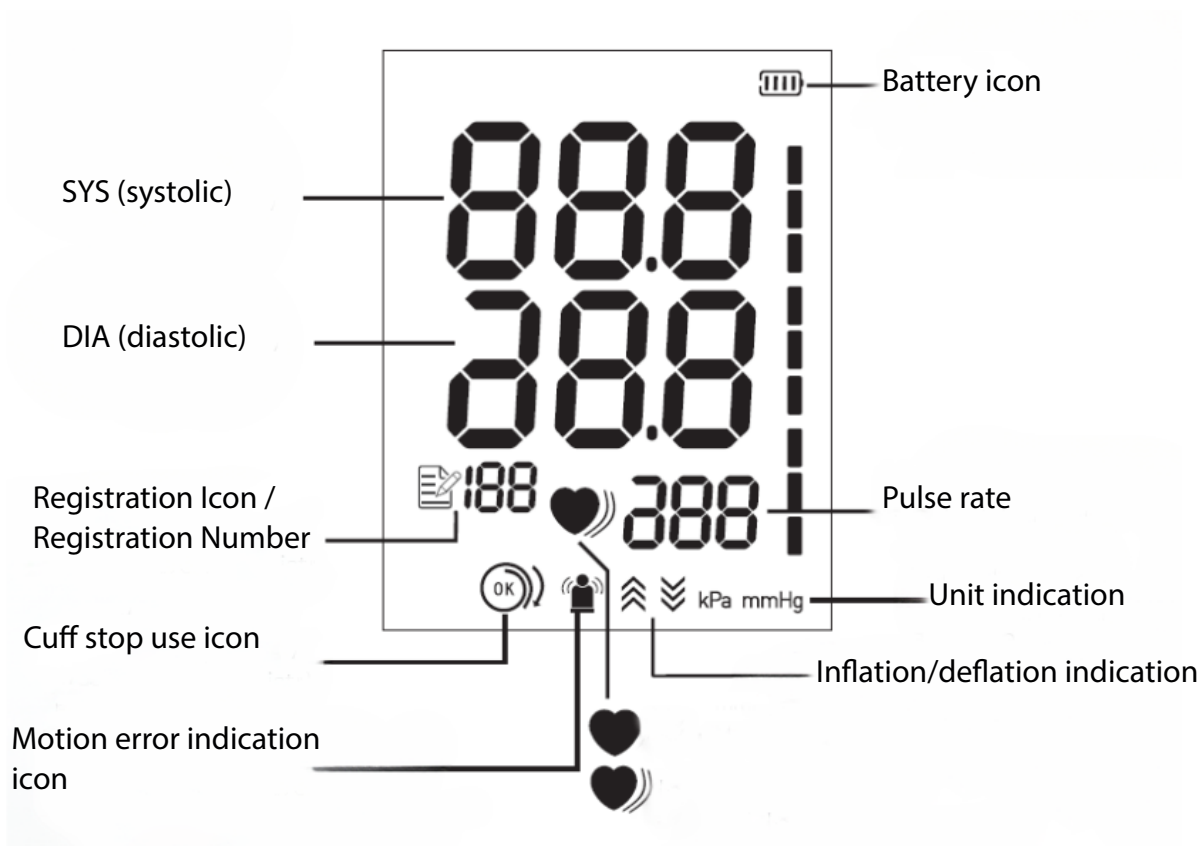5. INDICATIONS AND CONTRAINDICATIONS

INDICATIONS

This product is intended to measure blood pressure and pulse in adults over 12 years of age at home or in a medical facility.


CONTRAINDICATIONS

Except for newborns and pregnant women, everyone else can use our blood pressure monitor.

6. INSTRUCTIONS FOR USE

- Method of use of the cuff: Either arm can be measured. Do not measure other parts of the body.


1. Place the cuff 0,78"~1,18" above the elbow, keep the air tube inside the arm.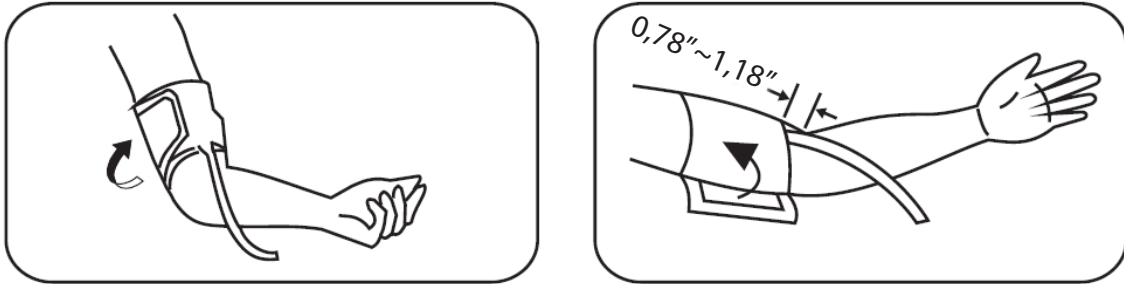


- Correct measuring posture:

Note: Any blood pressure measurement is influenced by the posture and physical condition of the person being measured.

1. Sit up straight and rest your arm on the table, palm up, and keep the centre of your fist and your heart at the same level. Also make sure that the air tube is not kinked.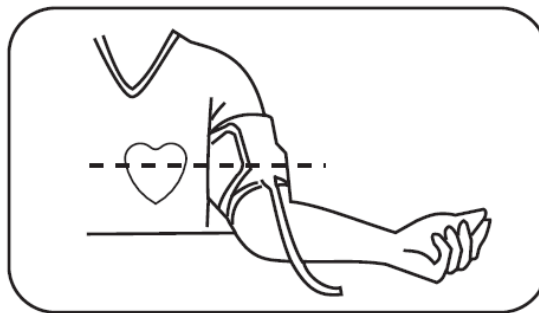


- Incorrect measuring posture:

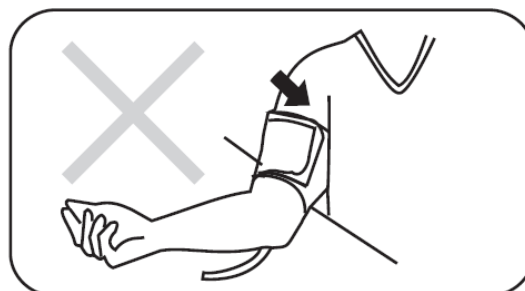1. Do not bend over or lean forward.
2. Do not sit cross-legged, feet should be flat on the floor.
3. Do not sit on a couch (abdominal pressure can increase blood pressure).
4. Do not rest your arm on the low table (may increase blood pressure).
5. Do not lie down.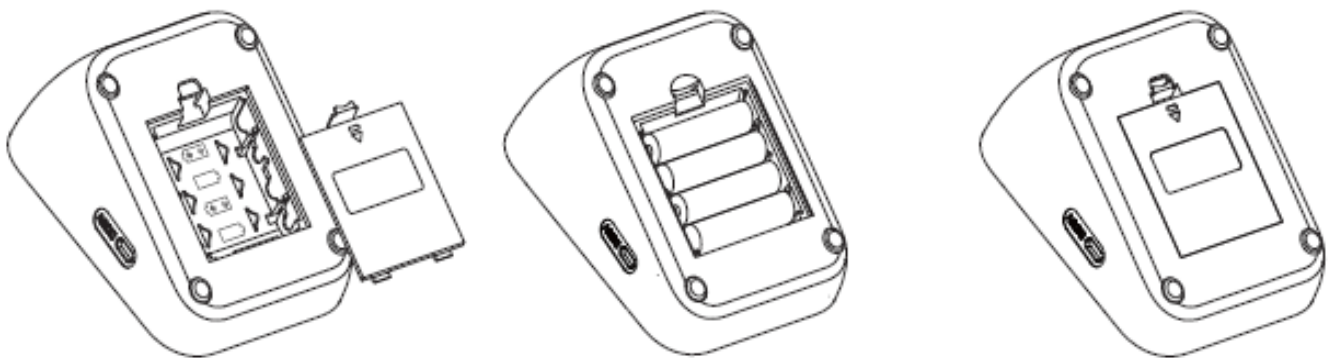

- Switching on: There are two ways to switch on the blood pressure monitor.
- Install AA batteries or plug the device into the mains via USB or using an adapter.

1. The "🔋" icon means that the battery is low.
2. The "🔋" icon means that the battery is running low. Please replace all batteries.
3. Remove the batteries if the monitor is not to be used for a long period of time (more than two months).

Note: An external 5V/1000mA DC power supply can be connected to the monitor.

- Start measuring: Remain seated for 5 minutes and press the "  " button to measure.
- Check the records: Press the "  " button to check the measurement records. This monitor can display the average of three measurements and store 90 records.

TIPS FOR USE:

- Sit still for 5 minutes before measuring to ensure a calm and stable state.
- Do not measure while standing, walking or with the body under pressure.
- Do not measure after smoking, drinking or taking caffeinated products.
- Do not measure after exercise or bathing.
- Do not talk or move during the measurement.
- Do not move, shake the arm or bend the fingers during measurement.
- Do not measure in extreme temperature conditions or in environments that change drastically.
- Do not measure in a moving vehicle.
- Do not measure continuously (an interval of 5 minutes should be left between two measurements), too frequent measurements may cause injury to the patient due to interference with blood flow. This may also provide inaccurate data.
- Do not measure within one hour after eating,
- Do not use the mobile phone near the device.
- In patients with arrhythmia, measurement results may not be accurate.
- Do not keep the cuff inflated for a long time.
- Note: Do not diagnose by measurement, follow your doctor's instructions.
- Declaration: The blood pressure monitor may not meet specifications if not stored within the required temperature and humidity ranges.
- Transport and storage conditions: The range is from -25°C to 35°C at a relative humidity up to 90%, non-condensing >35°C to 70°C at a water vapour pressure up to 5 kPa.
- Do not interconnect this appliance with other appliances not described in the operating instructions.
- In patients with arrhythmia, arterial sclerosis, poor perfusion, diabetes, pregnancy, pre-eclampsia, renal disease, patient movement, tremor, shivering, measurement results may not be accurate.


7. INSTRUCTIONS FOR USE

ADJUSTMENT OF THE UNIT:

- Press the "  " button to enter the unit setting.
- Press the "  " button to choose the unit between mmHg (most common) and kPa.
- Press the "  " button to finish.
- When you have finished setting the units, press the "  " button to switch off the device.


CONNECT SLEEVE:

- Connect the air connector of the cuff to the socket on the left side of the monitor.

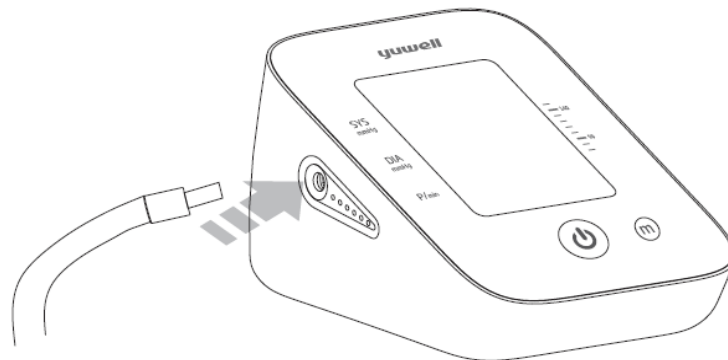USE OF THE CUFF METHOD:

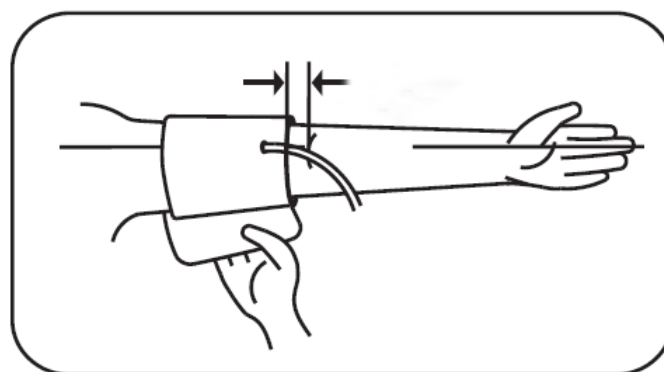Wrap the cuff around the upper arm (as shown in the picture). Keep the lower edge of the cuff in position above 0,78"-1,18" from the elbow joint and keep the air inlet that is inserted into the cuff on the inner side of the arm.


START MEASURING:


1. Remain seated for 5 minutes before measurement:

All icons on the screen appear for 1 second after pressing the "  " button and then disappear. After that, the deflate icon "  " flashes, which means that the system is in zero test.

After several seconds, the inflation icon "  " flashes, which means that the zero test is completed. Inflation then starts.

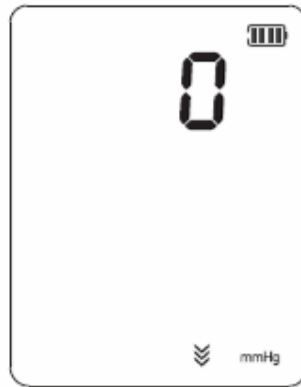Zeroing status

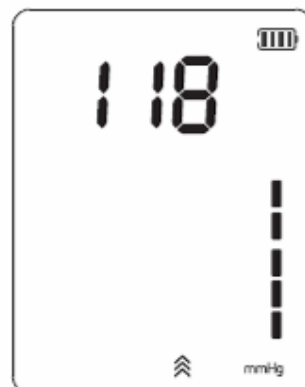Inflation status

2. Full screen display status:

The monitor starts measuring automatically after inflation is completed, and the measurement icon "  " starts flashing, the pressure release value starts decreasing gradually.

Please relax as much as possible and do not talk, talk or move during the measurement procedure.

MEASUREMENT COMPLETED:

After the measurement, the monitor will display the SYS (systolic pressure), DIA (diastolic pressure) and pulse rate, then the air will automatically deflate.

Display mmHg

Display kPa

REMOVE THE HANDLE: Press the "  " button to turn off. It will also turn off automatically in 3 minutes without operation.

OFF: Unplug the USB (or the adapter) to safely disconnect it from the mains (if it has been used in this way).

EMERGENCY STOP DURING MEASUREMENT: If it is necessary to stop the measurement for discomfort or other reasons, press the "  " button, the measurement will stop immediately and the air will be released quickly. Remove the cuff manually if the "  " button does not work.

MEMORIES: The monitor will store measurement data automatically (including blood pressure and pulse), and the upper limit of records is 90.

1. Press the "  " button to check the records.
2. Press the "  " button to display the average value of the last 3 measurements.
3. Press the "  " button again to display the 1st memory group, the serial number is displayed from "1" to "90". "1" is the last group and "90" is the oldest. The data 90 can be replaced by 89 when the memory capacity is full.
4. Read the recorded data by pressing the "  " button according to the sequence: "1", "2" "90" (max). Then go back to 1. Press and hold the "  " button to search the data quickly.
5. Press the "  " button to turn off the monitor.

Erasing recorded data

1. Press "  " and "  " buttons until the display shows the image, which means that the recorded data has been erased.
2. Press the "  " button to turn off the monitor after deleting.

SYSTEM RESTORATION

Press the "  " button after battery installation, then the display will show the "  " icon, which means that the system is in zero test. Several seconds later, the "  " icon disappears and the air pump starts to inflate at the same time, indicating that the test is complete. Then press the "  " button to stop inflation and remove the batteries to access the next step.

Note: You must restore the system before entering static mode, otherwise it may cause inaccurate results.


8. MAINTENANCE AND CLEANING

Please observe the following guidelines to protect the device and ensure accurate measurements:

- Store the blood pressure monitor and accessories properly after use.
- Do not leave the blood pressure monitor and accessories exposed to heat, moisture, dust or sunlight.
- The sleeve is equipped with an internal airbag, use it with care and do not bend, pull or twist it.
- Warning: Do not disassemble, repair or modify the appliance without authorisation.
- Do not carry out repairs or maintenance while the appliance is in use.
- If used by many people, clean the device with a soft cloth dampened with 75% ethanol, but do not allow the ethanol to get into the tensiometer and cuff.
- If used by one person, clean the device with a soft, dry cloth or a soft cloth dampened with a small amount of water, but do not allow water to get into the blood pressure monitor and cuff.
- Do not clean the appliance when it is connected to the mains.

9. SYMBOLS RELATING TO PRODUCT SAFETY REQUIREMENTS AND THEIR MEANING

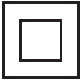	Caution				
	Type BF applied part				
	Follow instructions for use				
	Manufacturer		Importer		
	Date of manufacture				
	Authorised Representative of the European Community				
	Do not dispose of with household waste				
	Symbol indicating packing position		Fragile		Keep dry

IP20	IP Classification		
CE 0123	CE marking and notify body number		
	Double insulated (no earth wire required)		
UDI	Indicates a carrier that contains unique device identifier information		
	Direct current	DC	Direct current
	Alternating current		
SN	Serial number	LOT	Batch code
MD	Medical device		
	An item which poses unacceptable risks to the patient, medical staff or other persons within the MR (magnetic resonance) environment		

10. CONDITIONS OF GUARANTEE

This product is covered by a 3-year warranty against manufacturing defects.

Thank you very much for trusting Mobiclinic with our TENSIMAX upper arm blood pressure monitor! We value your trust in our products and are committed to always offering you the best quality.

If you have any questions or need further assistance, please do not hesitate to contact us through our customer service at **clientes@mobiclinic.com** . We appreciate your comments and suggestions for further improvement.

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CO., LTD.
GRUPO R. QUERALTÓ S.A.