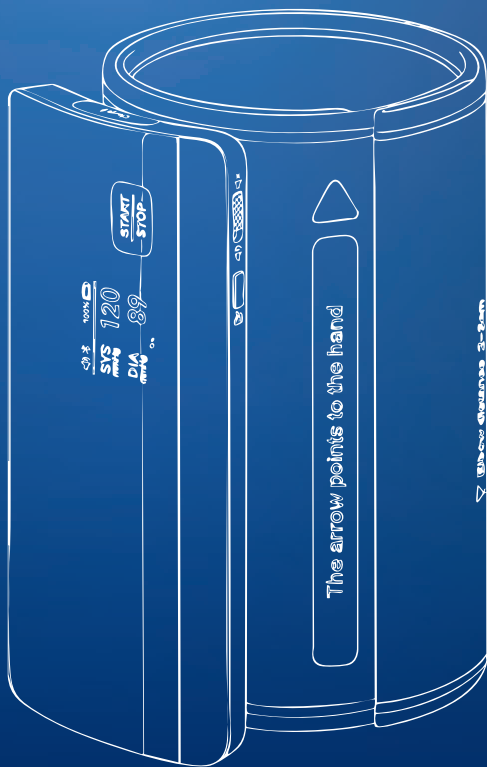
MANUAL DE INSTRUCCIONES

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BLUETOOTH

CARDIOMED

Manual de instrucciones

ES

Instruction manual

EN

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FR

Manuale D'Istruzioni

IT

Anweisungen

DE

Instruções

PT

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NL

Bruksanvisning

SW

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PL

Brugsanvisning

DA

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IMPORTANT

Keep the instructions for future reference. Read these instructions before assembling and using this product. Adult assembly is required.

Thank you for choosing the CARDIOMED Bluetooth blood pressure monitor! In this manual you will find all the information you need to assemble, use and maintain this product correctly and safely.

This blood pressure monitor and its included mobile app allow you to easily and accurately measure and monitor your blood pressure from the comfort of your home. Bluetooth connectivity synchronises the device with the app, where you can save and review your measurements at any time. It is an ideal tool to keep track of your cardiovascular health in a practical and accessible way.

Before you start, we recommend that you read this manual carefully to ensure optimal and safe use of the product.

As a guarantee of the quality of our products, the manufacturer has a management system certified under ISO 13485. And this product complies with the requirements of Regulation (EU) 2017/745 on medical devices.

Notice to the user and/or patient: any serious incident that has occurred in relation to the device must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is established.

1. WARNINGS AND PRECAUTIONS

- 1. WARNING!** Do not use the cuff on an injured arm or intravenous drip.
- 2. WARNING!** Do not use the device if the arm where intravascular access or therapy, or an arteriovenous (A-V) shunt is present due to temporary interference with blood flow as this may cause injury to the patient.
- 3. WARNING!** Do not use the cuff on the arm on the side of a mastectomy or lymph node cleansing.
- 4. WARNING!** Note that when the cuff is inflated, it will simultaneously lose the function of other monitoring devices on the same limb.
- 5. WARNING!** Make sure that the operation of the automated blood pressure monitor does not lead to a prolonged disturbance of the patient's blood circulation.

2. PURPOSES OF THE PRODUCT

- **Accurate blood pressure measurement:** Provides accurate and reliable readings for effective monitoring of your cardiovascular health.
- **Bluetooth connectivity:** Allows quick and easy synchronisation with the mobile app to store and review measurements.

- **Measurement history:** Record your results in detail for long-term monitoring.
- **Ease of use:** Intuitive design for easy and convenient use at home.
- **Accessibility from anywhere:** The mobile application allows you to review your results anytime, anywhere.
- **Continuous monitoring:** Ideal for those who need to monitor their blood pressure on a regular basis.
- **Data sharing with healthcare professionals:** Possibility to share your measurement history with your doctor for better follow-up.
- **Personalised care:** Provides key information to make informed decisions about your cardiovascular health.

3. PRODUCT FEATURES


PRODUCT NAME	Bluetooth blood pressure monitor
MODEL	CARDIOMED
MODEL REFERENCE	E-00072/55
PRODUCT DIMENSIONS	<ul style="list-style-type: none"> • With sleeve: 4,92"* 3,93"* 2,83" • Without sleeve: 4,92"* 2,44"* 0,94" • Sleeve 360°: 8,66"- 17,71"
MATERIALS	<ul style="list-style-type: none"> • Rear cover PC + ABS • PC top cover • Cuff: Velcro nylon and polyester cotton border fabric
OTHER CHARACTERISTICS	<ul style="list-style-type: none"> • Voice function • Bluetooth transmission • Powered by lithium battery • Capacity to store up to 50 measurements

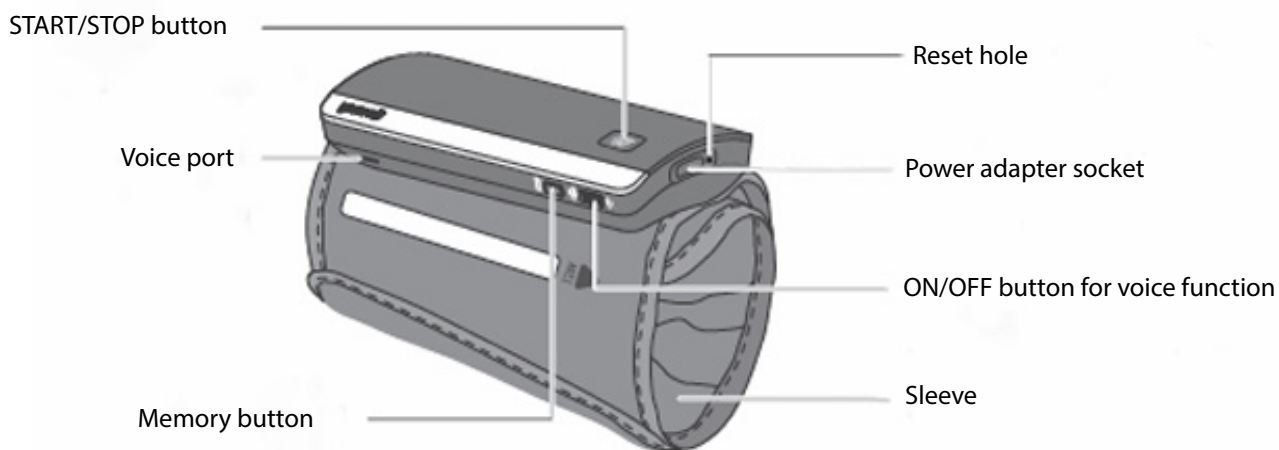TECHNICAL PARAMETERS:

RELATIVE PRINCIPLE	OSCILLATION MEASUREMENT	
PRESSURE MEASUREMENT	Range	SYS: 60~260 mmHg DIA: 40~210 mmHg Cuff pressure: 0 mmHg-300 mmHg
	Accuracy	Within ± 3 mmHg (± 0.4 kPa)
PULSE RATE MEASUREMENT	Range: 40~200 beats/min	
	Accuracy: Within 5% of reading value	
WORK SYSTEM	Continuous operation	
TECHNICAL CLASSIFICATION	Class II, Type BF applied part (the sleeve is the applied part)	
PRESSURE SENSOR	Semiconductor pressure sensor	
PRESSURISATION	Automatic pressurisation	
DEPRESSURISATION	Automatic air release	
POWER SUPPLY	Battery	 3,7V
	Adapter	<ul style="list-style-type: none"> Input ~ 100-240 V, 50/60 Hz, 0.35 A Output:  1A

BATTERY USAGE TIMES	The battery can be used about 250 times with a full charge
DIMENSION	Approx. 4,92"* 2,44"*0,94" (without sleeve)
WEIGHT	Approx. 0,56 lb (without sleeve)
IP RATING	IP22: Against ingress of water with harmful effects: dripping (15° inclined) / Against ingress of solid objects foreign objects: ≥ 0,49" in diameter.
HANDLE	8,66" - 17,71"
SHELF LIFE	5 years (6 times per day) for the blood pressure monitor, 5.000 times for the cuff

4. PARTS AND COMPONENTS

- Accessories: USB cable
- Screen icons:

ICONS	DESCRIPTION
	Bluetooth icon: This icon appears when Bluetooth is connected.
	Voice activation icon: When the voice function is activated, this icon appears.
	Voice disable icon: When the voice function is disabled, this icon appears.
	Cuff positioning detection icon: When the cuff is not sufficiently tightened, this icon appears. Please fasten the cuff correctly and measure again.
	Cuff positioning detection icon: When the cuff is correctly positioned, this icon appears.
	Motion error indication icon: When incorrect body movement is detected during measurement, this icon is displayed.
	Deflation icon: When the cuff is deflated, this icon appears.
	Heartbeat icon: When a pulse is detected, this icon appears.
	Irregular heartbeat indication icon: When an irregular pulse signal is detected during measurement, this icon appears.
	Charging icon: When charging, this icon appears.
	Battery icon: Displays the remaining battery power.

5. INDICATIONS

This product is intended to measure blood pressure and pulse in adults over 12 years of age at home or in a medical facility.

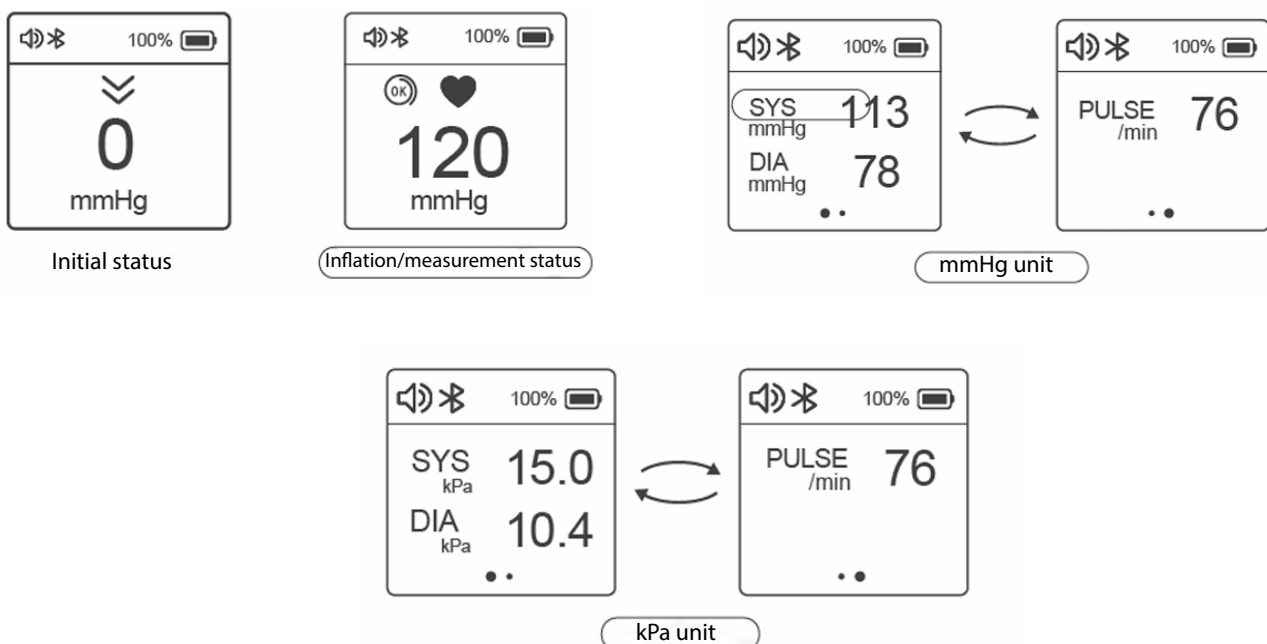
6. INSTRUCTIONS FOR USE

BLOOD PRESSURE MEASUREMENT:

- Please pay attention to the following points before measuring:
 - Remain seated for 5 minutes before measurement.
 - Measurements should be taken at the same time every day.
 - Do not eat, smoke, drink, bathe or engage in high-intensity sports in the thirty minutes prior to measurement.
 - For multiple measurements, the time interval between two adjacent measurements shall be at least 5 minutes or more.
 - Estimate your blood pressure status according to the blood pressure classification chart and consult your doctor.

1. Press the [Start/Stop] button: The display shows the deflate icon " ∨ " after pressing the [Start/Stop] button. Then the deflate icon disappears and the air pump starts to inflate.

2. Measurement process: The tensiometer starts the measurement automatically during inflation. After the measurement is finished, the tensiometer will automatically disconnect the air.

NOTE!

- If an unexpected reading is obtained, take another measurement after a 5-minute break or consult your doctor.
- The upper air inflation limit pressure is 300 mmHg/40.0 kPa. Do not maintain the inflated state for a long time to avoid damage.
- If it is necessary to stop the measurement for discomfort or other reasons, press the [Start/Stop] button. The measurement will stop immediately and the air will be released quickly. Remove the cuff manually if the [Start/Stop] button does not work.

Detection of incorrect cuff positioning: When the cuff is correctly positioned, the "  " icon appears. When the cuff is loosely positioned, the "  " icon appears.

Press the [Start/Stop] button to stop the measurement when the "  " icon appears, and measure again after the cuff is correctly positioned.

Motion error indicator: When an incorrect body movement is detected during measurement, the "  " icon appears. Please measure again after 5 minutes of waiting.

Irregular heartbeat indication: It means that the measurement result may be inaccurate when the "  " icon appears. Please measure again. If the icon appears for several measurements, please consult your doctor.

About Bluetooth transmission: To ensure that measurement results can be transmitted via Bluetooth, maintain a maximum distance of 393,70" between the device and the mobile phone.

If you want your blood pressure measurements to be sent to your phone via Bluetooth, get a phone that supports Android 5.0 or higher, or iOS 9.0 or higher to download **Yuwell HealthCare+**.

Yuwell HealthCare+
Medicina

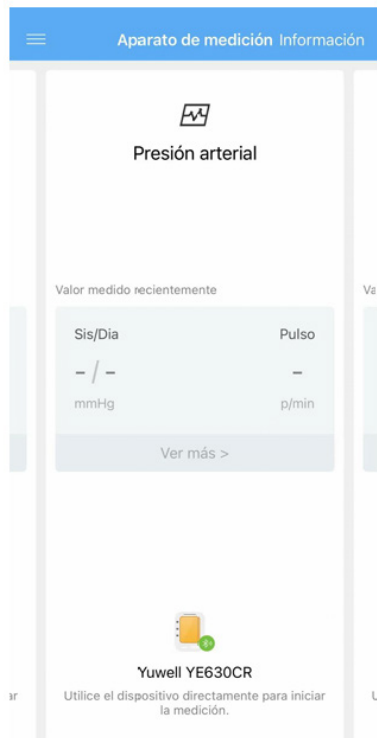
Once you have installed the APP, all you have to do is activate Bluetooth on your mobile device and open the APP. When you are going to measure your blood pressure, leave the APP open and you will see how the data is recorded.

Here is what you will see when you do not have bluetooth on your mobile device:

and this will come up when you have the bluetooth correctly turned on in your mobile device:

Once it is correctly connected, press the Start/Stop button and it will start measuring. When the measurement is finished, the results will look like this:

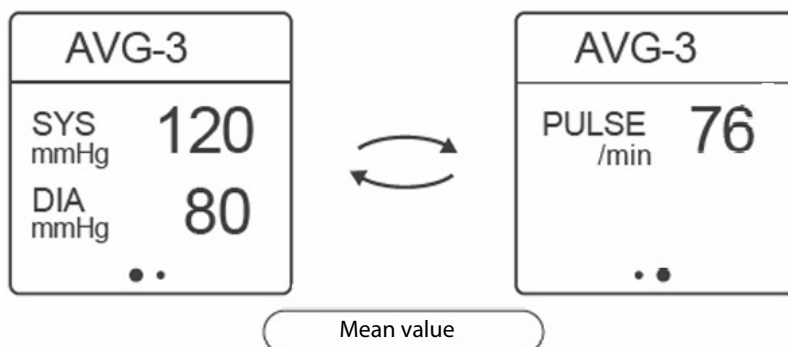
Remove the cuff once deflated.

Then press the [Start/Stop] button to turn off. It will turn off automatically in 3 minutes without any operation.

CHECK THE MEMORY STORAGE DATA:

The blood pressure monitor will store measurement data automatically (including blood pressure and pulse). The maximum number of records is 50. The 50th data will be replaced by the 49th and the 1st data will be replaced by new data when the memory capacity is full.

1. Press the [Memory] button for the first time to check the average value of the last 3 measurements.
2. Press the [Memory] button again to check the 1st memory group.
3. Press and hold down the [Memory] button to quickly check the relevant memory data.
4. Press the [Start/Stop] button to turn off the blood pressure monitor.

DELETE DATA FROM MEMORY STORAGE:

1. Press and hold down the [Start/Stop] button and the [Memory] button simultaneously, until the display shows the image, indicating that the recorded data has been deleted.
2. Then press the [Start/Stop] button to turn off the blood pressure monitor. Note: This operation will erase all recorded data.

**FUNCTION SETTING:****1. Setting the time and date:**

- Press the [Start/Stop] and [Memory] buttons simultaneously for more than 3 seconds until the year number starts flashing.
- Press the [Memory] button to set the year.
- When the year setting is completed, press the [Start/Stop] button to switch to the month setting.
- Use the same method to set the month, day, hour and minute in that order.

2. Language setting:

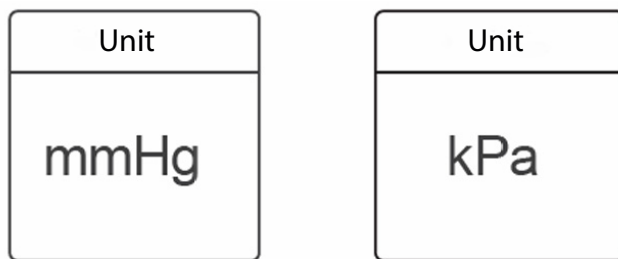
When the time and date setting is finished, press "  " button to enter the language setting. Press "  " to select the desired language.

3. Voice volume adjustment:

When the time and date setting is completed, press the [Start/Stop] button to enter the voice volume setting. Press the [Memory] button to adjust the voice volume. Tip: When the voice function button is off, there will be no voice output function.

4. Adjustment of the unit:

When the voice volume setting is completed, press the [Start/Stop] button to enter the unit setting. Press the [Memory] button to select the unit between mmHg (this is the most common unit of measurement) and kPa. Press the [Start/Stop] button to finish the setting.

TIP: In kPa unit mode, there are no voice prompts

ALARM SYSTEM:

When the determined blood pressure is outside the nominal range, a visual alarm signal will appear on the display. Refer to the table below for details.

ALARM INDICATION	SCREEN CONTENT AND CAUSES
	<p>1. When "Hi" appears in the SYS area, it indicates that the systolic blood pressure measurement is above 260 mmHg.</p> <p>2. When "Hi" appears in the DIA, indicates that the pressure measurement diastolic blood pressure is greater than 210 mmHg.</p>
	<p>1. When "Lo" appears in the SYS area, it indicates that the systolic blood pressure measurement is less than 60 mmHg.</p> <p>2. When "Lo" appears in the DIA, indicates that the pressure measurement diastolic blood pressure is less than 40 mmHg.</p>

When the above alarm indication appears, take another measurement or consult your doctor.

- If the alarm indication cannot be resolved or the user feels uncomfortable, consult a physician as soon as possible.
- If the alarm indication cannot be resolved or you need to verify the functionality of the alarm system, consult the manufacturer.

7. MAINTENANCE AND CLEANING

Please observe the following guidelines to protect the device and ensure accurate measurements:

- Store the blood pressure monitor and accessories properly after use.
- Do not leave the blood pressure monitor and accessories exposed to heat, moisture, dust or sunlight.
- The sleeve is equipped with an internal airbag, use it with care and do not bend, pull or twist it.
- **Warning:** Do not disassemble, repair or modify the blood pressure monitor without authorization.
- Do not carry out repairs or maintenance while the appliance is in use.
- If used by many people, clean the device with a soft cloth dampened with 75% ethanol, but do not allow the ethanol to get into the tensiometer and cuff.
- If used by one person, clean the device with a soft, dry cloth or a soft cloth dampened with a small amount of water, but do not allow water to get into the blood pressure monitor and cuff.
- Do not clean the appliance when it is connected to the mains.

Recovery time:

- When the tensiometer has been exposed to extreme temperatures (-20°C or 50°C), the time required for it to return to an optimum temperature for use is 2 hours at an ambient temperature of 20°C.

Operating and storage conditions:

- Operating conditions: Temperature range + 5°C to + 40°C; relative humidity range 15% to 90%, non-condensing; and atmospheric pressure range 70 kPa to 106 kPa.
- Transport and storage conditions: The range is from -25°C to 35°C at a relative humidity up to 90%, non-condensing >35°C to 70°C at a water vapour pressure up to 5 kPa.
- Operating Environment: Avoid electromagnetic interference, violent shocks and noisy environment.

8. PROBLEM SOLVING

UNUSUAL NOISES	CAUSES	SOLUTIONS
Err3	The pressure value does not reached 5 mmHg in 4 seconds.	Please tighten the new sleeve and measure again.
Err4	It is not possible to measure the pressure.	Hold the arm and the body still and come back to measure.
Err5	Pressurisation error.	Please tighten the sleeve well and measure again.
Err6	Pressurisation error-caused by arm or body movement.	Hold the arm and the body still and come back to measure.
Err7	The sleeve is too loose or falls off.	Please tighten the sleeve well and measure again.
Err8	Pressure exceeds the maximum value (300 mmHg).	Hold the arm and the body still and come back to measure.
Low battery 	The battery is running low.	Connect the power adapter for charging.
Other	If the blood pressure monitor does not work after pressing the [Start/Stop] button, check if the battery has sufficient charge.	

About blood pressure measurement :

- **Why is it important to be comfortable and relaxed before measuring your blood pressure?** - If you stretch or strain your arm forcefully, this can cause an increase in blood pressure.
- **Why is it necessary to check whether we have taken blood pressure medication before the measurement?** - If you have taken antihypertensive medication, its effect will diminish after a few hours and the blood pressure will rise accordingly. Please consult your doctor for more details.
- **Why is it essential to ensure that the cuff is fitted correctly?** - If the cuff is worn too tight or too loose, the measured data may be inaccurate.
- **Why keep the centre of the cuff and the heart at the same height?** - If the measured area is above or below the heart, the data obtained may be inaccurate.
- **Why not move or talk during the measurement?** - If you move or talk while taking the measurement, the data may be inaccurate.
- **Why should there be an interval of at least 5 minutes or more between two consecutive measurements for multiple measurements?** - Because repeated measurements can lead to blood pooling in the arm, which can cause numbness and incorrect measurement data.
- **Why can the data vary from measurement to measurement?** - Because blood pressure is constantly changing:
 - Generally, blood pressure rises in winter and falls in summer.
 - Poor sleep can cause a slight increase in blood pressure.
 - Note: Measurements should be taken at the same time each day.

9. INFORMATION ON ELECTROMAGNETIC COMPATIBILITY

BASIC BENEFITS

1. Pressure gauge error limits: Over the temperature range of 5°C to 40°C and the relative humidity range of 15% to 90% (non-condensing), the maximum error for the HOSE pressure measurement at any point in the NOMINAL range of the measurement shall be less than or equal to ± 3 mmHg (± 0.4 kPa) of the reading.
2. Reproducibility of BREATHING PRESSURE DETERMINATION: The laboratory reproducibility of the AUTOMATED TENSIO METER BREATHING PRESSURE DETERMINATION shall be less than or equal to 3,0 mmHg (0,4 kPa).

NOTE:

- Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should not be used within 30 cm (12 inches) of any part of the TENSIO METER, including any cables specified by the manufacturer. Failure to do so may result in degradation of the performance of this equipment.

- This equipment may not provide adequate protection for radio frequency communication services. The user may have to take mitigating measures, such as relocating or reorienting the equipment.
- Do not place or stack the instrument near or on top of other instruments while in use. If it must be placed near or on top of other instruments, check and verify that the instrument is operational.

WARNING: The Operator must not use the system and must inform customer service, if the ESSENTIAL PERFORMANCE is lost or degraded due to EM DISTURBANCES.

WARNING: The use of accessories and cables other than those specified or supplied by the manufacturer of this equipment may result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and may cause malfunction.

The list of cables and accessories is as follows:

CABLE NAME	CABLE LENGTH	CABLE SHIELDING
DC power cable	≤47,24"	Unshielded

There is a potential risk of radio frequency interference between the device and other devices. If so, find out what the problems are and take the following measures:

1. Switch the device off and then on again.
2. Change the orientation of the device.
3. Keep the product away from interfering devices.

10. CONDITIONS OF GUARANTEE

This product is covered by a 3-year warranty against manufacturing defects.

Thank you for trusting Mobiclinic with our bluetooth blood pressure monitor CARDIOMED! We value your trust in our products and are committed to always offering you the best quality.

If you have any questions or need additional assistance, please do not hesitate to contact us through our customer service at clientes@mobiclinic.com. We appreciate your comments and suggestions for further improvement.

Thank you for joining Mobiclinic!

0123 Made in P.R.C.

JIANGSU YUYUE MEDICAL EQUIPMENT & SUPPLY
CO., LTD.

No.1 Baisheng Road Development Zone, 212300
Danyang, Jiangsu, China
sales@yuyue.com.cn

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GRUPO R. QUERALTÓ S.A.